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
# RATIONALIZED ROOT TREATMENT

Millions of teeth have been treated successfully with this **easy, modern and scientifically** tested Endodontic Method applicable for **every case** of root treatment. Two preparations, N2 Universal for treatment of vital teeth and N2 Apical for treatment of gangrenous teeth, are used.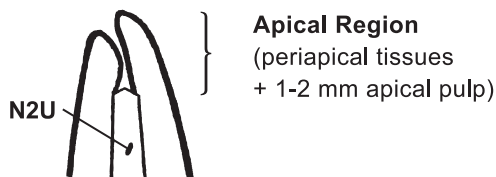

**For dependable, predictable results in treating vital teeth, respect the integrity of the apical region!**

## Fundamental Principles

### in treating vital teeth

Do not probe the apical region with small instruments. Barbed nerve extractors should not be used, they lacerate the apical pulp and cause bleeding.

A **reamer** of approx. the same diameter as the apical part of the canal exercises friction against the canal walls, thus avoids penetration of the apical region and gives a **clean cut** of the pulp.


In treating vital teeth, N2U paste should not be forced beyond the apex. The apical region in vital cases is intact and should not be irritated by extrusion of a filling material. However, small quantities of N2U accidentally forced beyond the apex are well tolerated by the apical tissues even if a transitory reaction may ensue.

**Pulp bleeding** is always the consequence of using too small instruments. To stop hemorrhage insert N2U paste into the canal with a spiral. At the same, or the subsequent, sitting ream again and fill permanently with N2U.

### How to prevent reactions

Do not fill the canal at the first sitting in cases of uncertain diagnosis, persistent bleeding, suspected perforation, etc. Instead, place a small quantity of N2U paste on the floor of the pulp chamber, take a control X-ray. At the next sitting ream the canal again and permanently fill with N2U.


## VITAL TEETH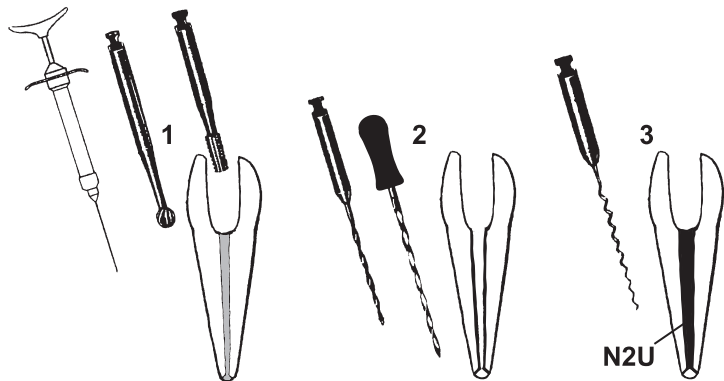

### Treatment in one Sitting


- 1 Local anesthesia. Open pulp chamber.
- 2 Ream the canal up to 1-2 mm from the apex. Use a reamer the same size as the apical part of the canal.
- 3 Fill permanently with N2U.

In molars with curved roots or difficult access, only ream as far as **reasonably possible**. The reamable portion of the canal must be well enlarged.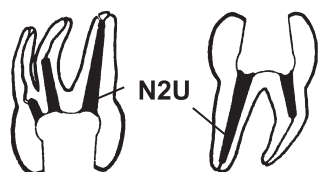

**NOTE:** In treating vital teeth N2U should not be forced into the intact apical region.

## GANGRENOUS TEETH

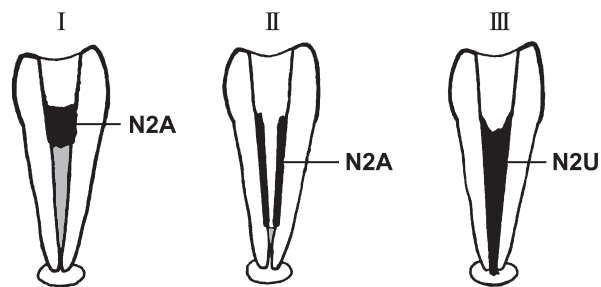1. **Sitting** Clean pulp chamber.  
Ream 1/3 of the canal without pressure (see Fig. I).  
Insert **Apical** into the pulp chamber.  
Close crown with sealing materials.  
**Interval:** one or two weeks.
2. **Sitting** Gently ream 2/3 of the canal.  
Do not probe apical region.  
**Parietal Dressing** (see Fig. II).  
Close crown with sealing materials.  
**Interval:** one or two weeks or longer.
3. **Sitting** Ream to the apex.  
Fill permanently with N2U (see Fig. III).

**NOTE:** In treating **gangrenous teeth** a small quantity of N2U may be extruded into the granuloma (N2U stimulates healing process).

**Parietal Dressing.** Only the canal walls are coated with a thin film of paste applied with a reamer. The canal center remains empty. Don't use cotton fibers or paper points together with N2U; the paste hardens in the canal and the dressing would be difficult to remove.

## Treatment of DECIDUOUS TEETH

### Vital teeth

Remove the coronal pulp only. Cover the pulp stump with N2U or N2A paste. Close with sealing material. At the next sitting, coronal restoration is applied directly upon the hardened N2U.

### Gangrenous deciduous teeth

Clean pulp chamber and canal opening. Gently enlarge the coronal third of the canal **without probing the apical region**. Insert N2A paste with a spiral. Close with sealing materials. At the next sitting, coronal restoration is applied as usual.

### ACTIVE INGREDIENTS OF N2 PREPARATIONS (N2U, N2A)

Paraformaldehyde, Bismuth salts, Zinc oxide, Eugenol and others.

### HOW TO MIX:

Add 0.3g of Powder to 5 drops of Liquid and spatulate to a paste of medium consistency

**AGSA JAPAN CO.,LTD.**